


VACUTEST® SAMPLING INSTRUCTIONS



For safe and quick transfers of urine samples from 120ml Urine Collection Containers to the Vacutest Urine Collection Tube (4ml, 9ml or 9.5ml)

Please read and follow the instructions below carefully.

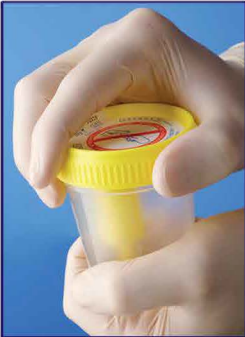

OPEN THE CAP BY UNSCREWING ANTI-CLOCKWISE.



LAY THE CAP UPSIDE DOWN ON A CLEAN SURFACE .




DO NOT TOUCH INTERNAL SURFACES OF THE CONTAINER AND CAP.



COLLECT THE URINE SAMPLE. FILL THE CONTAINER UP TO ¾ OF THE CAPACITY.



REPLACE THE CAP AND TURN TIGHTLY IN A CLOCKWISE DIRECTION TO SEAL.

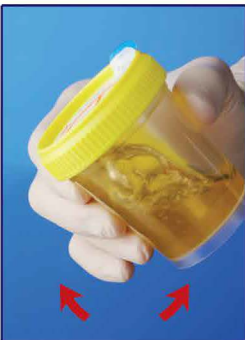

GENTLY SHAKE THE SAMPLE.



PARTIALLY RAISE THE PROTECTIVE LABEL (DO NOT remove it completely).



INSERT THE SAMPLE TUBE AND APPLY GENTLE PRESSURE. KEEP THE TUBE CONNECTED UNTIL IT IS FULL (end of flow).



REMOVE THE SAMPLE TUBE AND FULLY RESTICK THE PROTECTIVE LABEL.



WHEN USING SAMPLE TUBES WITH PRESERVATIVE: GENTLY SHAKE THE TUBE 8-10 TIMES.

If you have any questions relating to the Vacutest closed urine collection system, please do not hesitate to contact Elkay: sales@elkaylabs.com



A CALIBRE SCIENTIFIC COMPANY

